



Yes, you can.®

➤ Invacare® **Perfecto₂**™

NEW

Home Oxygen without Compromise

The stylish new *Invacare*® **Perfecto₂**™ oxygen concentrator is designed to exceed the requirements of the patient and provider in the ever demanding home oxygen market. The latest innovations in power and sound management result in lower running costs and increased patient comfort. The new package blends discreetly into the patient's environment whilst fulfilling prescriptions up to 5 liters per minute. With no component maintenance required for 5 years the **Perfecto₂**™ combined with **HomeFill**® II is the lowest 'Total Life Cost' home oxygen solution.





➤ **Whisper quiet performance**

Compact and quiet, the stylish new *Invacare® Perfecto₂™* can be easily integrated into the home environment. Innovative techniques have reduced the compressor and purge sounds to an almost undetectable level. Your peaceful environment will be yours to enjoy with the knowledge that you are receiving an uninterrupted supply of home oxygen.

➤ **Low power consumption**

Intelligent low power management software adjusts the internal system pressures according to the output demands. For the average user this equates to an additional 20 Watts per hour saving below the already low 300 Watt average consumption. With *Perfecto₂™* low power consumption, *Invacare®* is pioneering power saving technology together with high output performance.

➤ **Uncompromised reliability**

With no compressor rebuild for at least 5 years you have the peace of mind that you will receive the oxygen you need when you need it. Thanks to the latest advances in sieve material and compressor technology *Perfecto₂™* will deliver you the lowest 'Total Life Cost' home oxygen concentrator.

➤ **Compact Package**

Thanks to its state of the art design, the *Perfecto₂™* is the smallest 5 liter concentrator on the market. The size and performance of the compressor and sieve beds are not compromised in this new package. The user friendly controls and familiar safety features you have come to expect from the market leader have all remained in this new ergonomic, patient preferred design.



*Contact *Invacare®* for more information on *HomeFill® II*.

Options and Features



Power Saving

Looking after the environment with intelligent low power management software.



Reliable oxygen supply

No compromise: The full size compressor has no maintenance requirements for 5 years.



Easy to maintain

Filter maintenance is performed through the rear access door.



User friendly design

Conveniently located, easy to see humidifier bottle holder. Easy to access controls and display.



HomeFill® II compatible

Benefit from the independence and convenience of filling your own lightweight cylinders (Model: IOH200AW)



Pediatric Flowmeter


Adapt to low flow patients with no permanent modification using PreciseRx remote delivery device. Adjustable flow from 50cc to 750cc per minute. (Model: IRCPFI6AW)

Technical Specifications


Performance

Adjustable Flow	0.5 – 5 l/min
Oxygen concentration (30 min operation)	0.5 – 5 l/min 87 – 95.6 %
Oxygen outlet pressure	0.34 bar +/- 10% (34.5 kPa +/- 3.45 kPa)
Oxygen outlet temperature	+3° C / ambient temperature
Sound Level	<37 dB (A)
Battery power-fail alarm	No
Low oxygen flow sound alarm	< 0.5 l/min
Concentration alarms	> 85% Green light > 73% Yellow warning light < 73% Red warning light with sound
Self-diagnostic system	Failure code with memory
Compressor Life	No rebuild for 5 years
HomeFill® II compatible	Yes
Pediatric Flowmeter	Yes – No modification required (Model: IRCPFI6AW)


Technical data

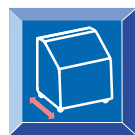

30,5 cm

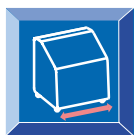

38 cm

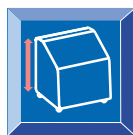

58,5 cm



20,5 kg



230 V, 50 Hz
(±10%)



280 W @ ≤ 3 l/min
max. 300 W

This product complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2002/96/CE



All information quoted is believed to be correct at time of print.
Invacare® reserves the right to alter product specifications without prior consultation.



INVACARE® INTERNATIONAL Sàrl
Route de Cité-Ouest 2 · 1196 Gland, Switzerland
Phone: +41 22 354 60 10 · Fax: +41 22 354 60 11
E-mail: hqeuropa@invacare.com

